



OINKMENT[®] *Spray*


INSTRUCTIONS FOR USING OINKMENT SPRAYERS

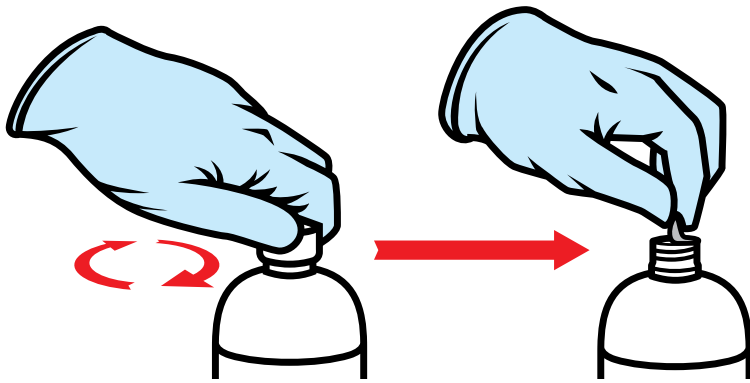

1. Open the bottle by unscrewing the cap and carefully remove the aluminum seal.

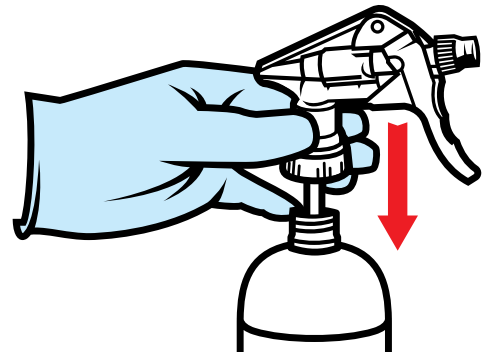


2. Insert a sprayer in the bottle and close it tightly.

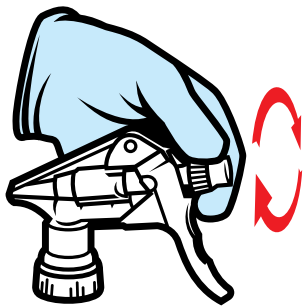

3. Unlock the nozzle tip of the sprayer by unscrewing it counterclockwise.



4. For best results, unscrew the nozzle tip one full circle "360 degrees".

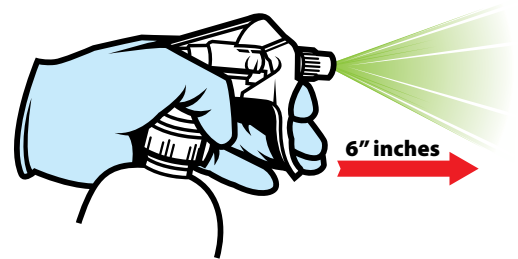

5. Apply **Oinkment** with spray tip approximately 6 inches from the wound, spraying to completely cover the wound and overlap onto surrounding healthy skin.



6. When application is completed, tighten the tip nozzle by screwing it clockwise, back to its initial position, so the product from the nozzle chamber will be released.

***Important:** Make sure after application to screw back the nozzle tip, releasing the excess of remaining product from the nozzle chamber, and avoiding the product to dry inside sprayer tip which can lead to sprayer clogging.

SCAN FOR



VIDEO

